




#### Kit Contents

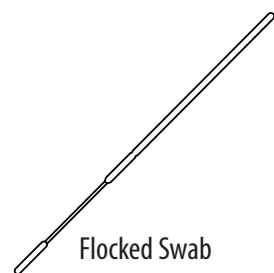

Flocked Swab

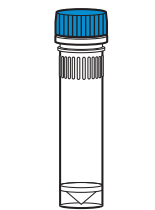


Dilution Buffer

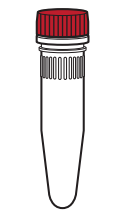

Lysis Buffer



Reaction Tubes

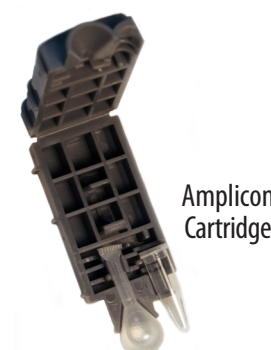


Amplicon Cartridge



Detection Chamber



#### Equipment

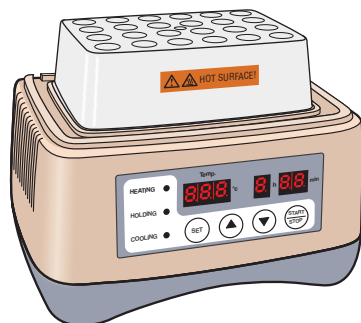


95° Lysis Block

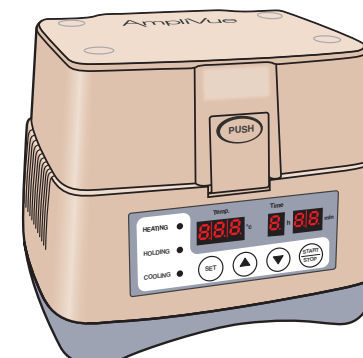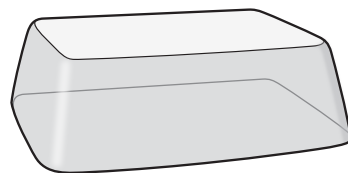


64° Amplification Block with heated lid


#### Assay Procedure - Heat Lysis

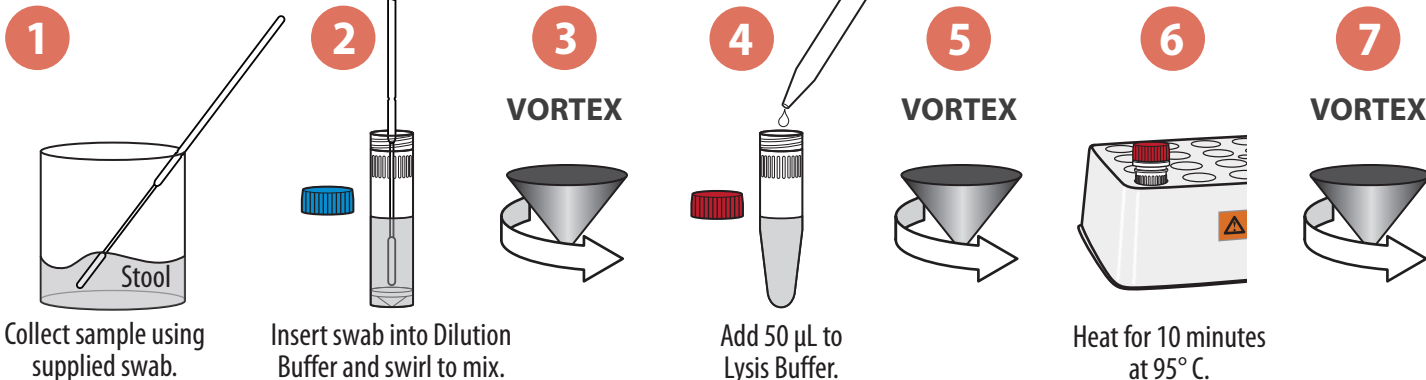

#### Amplification

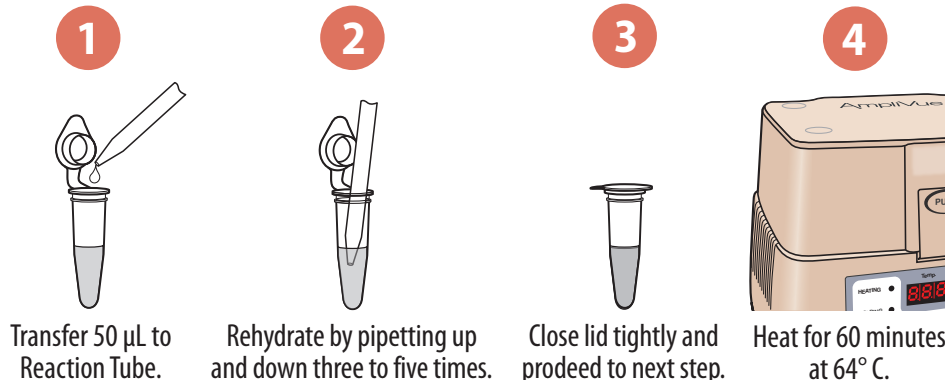


#### Detection

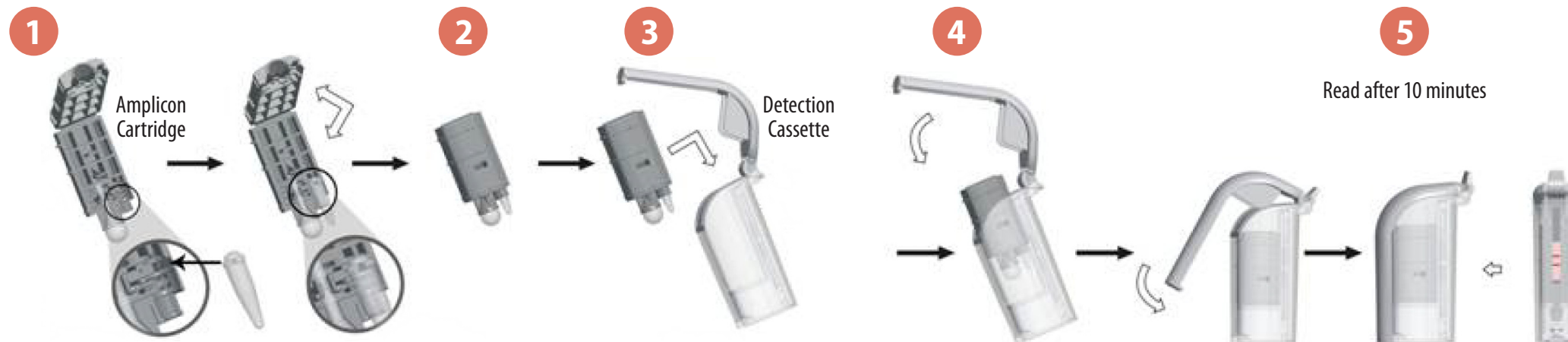


#### Interpretation of Results

**Toxigenic**  
*C. difficile*

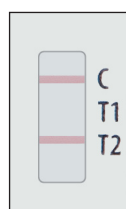

Toxigenic *C. difficile* DNA detected.

**No Toxigenic**  
*C. difficile*

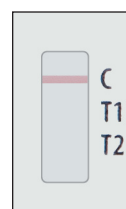

No Toxigenic *C. difficile* DNA detected.

**Unresolved**



Unresolved: failure due to inhibitory specimen, or reagent failure, or device failure. Repeat test with original stool specimen.